
Etac Rex

Manual

etac®

78570B 16-04-28

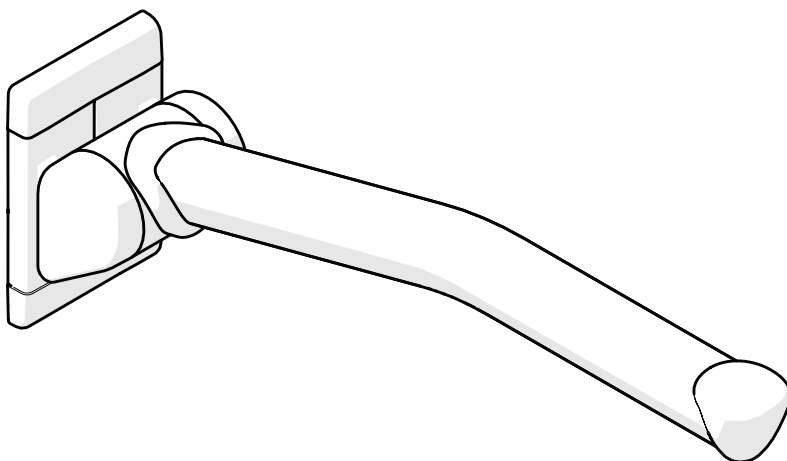
sv no da en de fi fr nl it es pt


A


Etac Rex 60 cm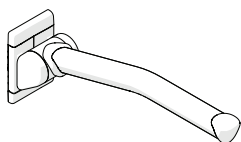


Etac Rex 70 cm



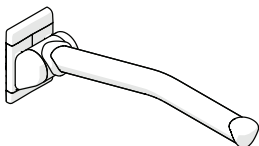
Etac Rex 85 cm

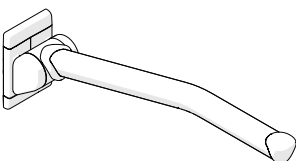Etac Rex 70 cm

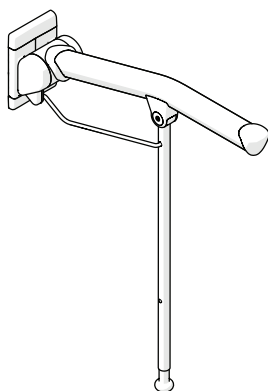
Etac Rex 85 cm

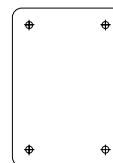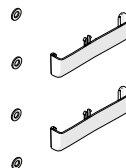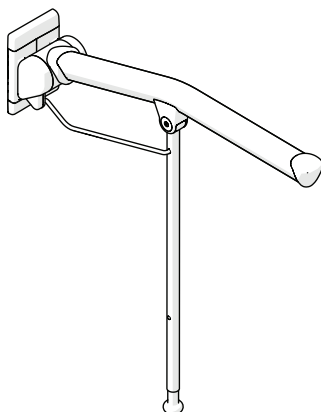
B

1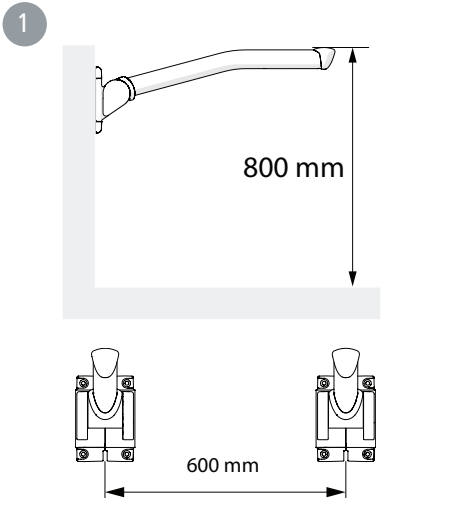


2

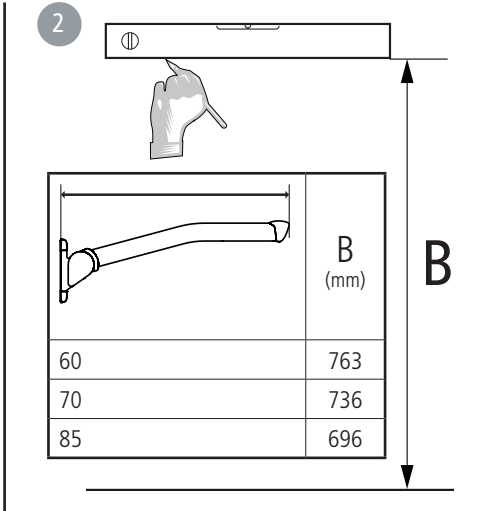
	B (mm)
60	763
70	736
85	696


3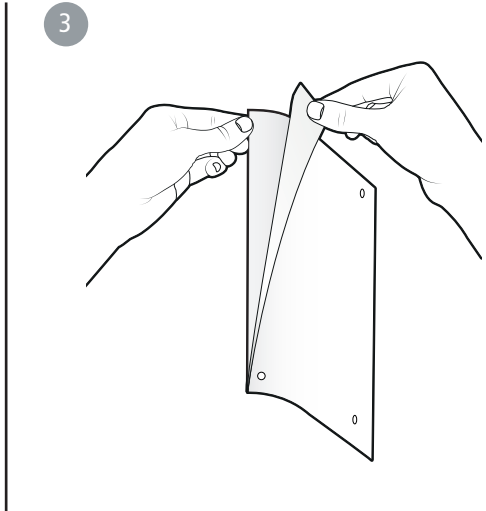


4

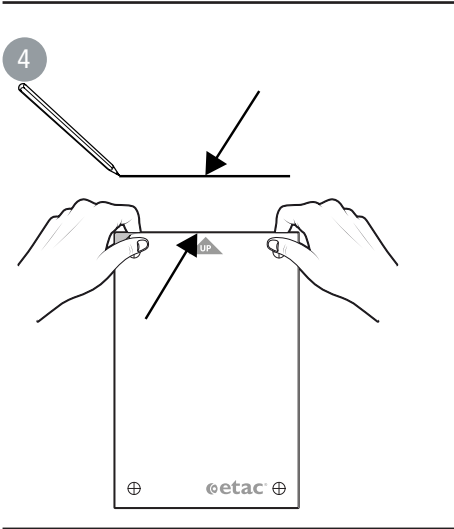
5

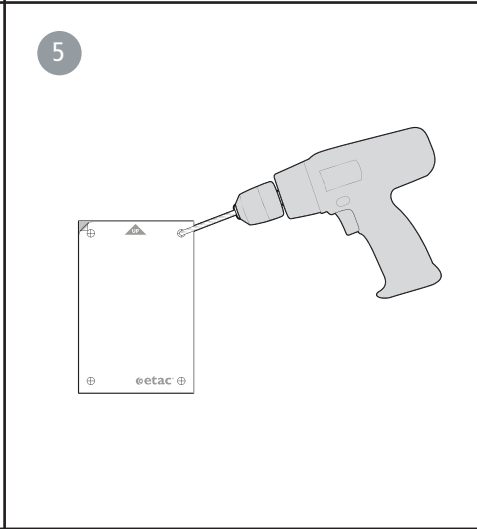
6

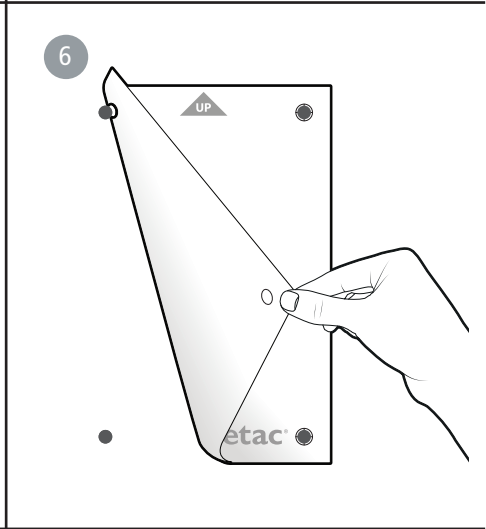
7

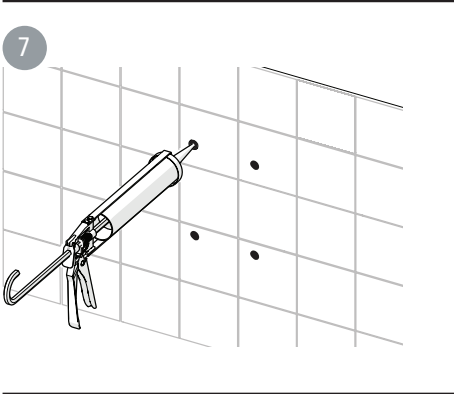
8

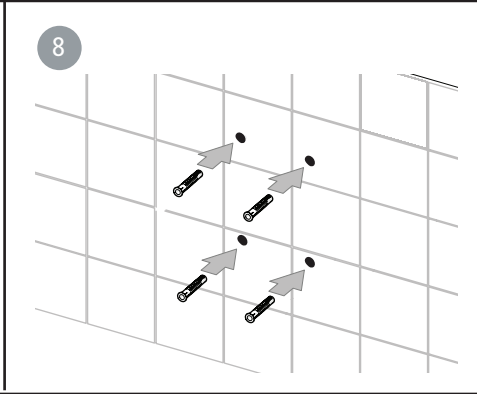
9


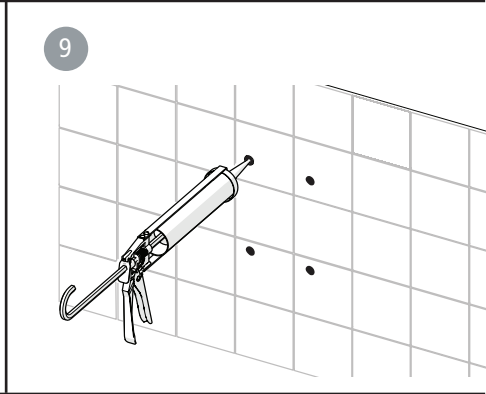
10

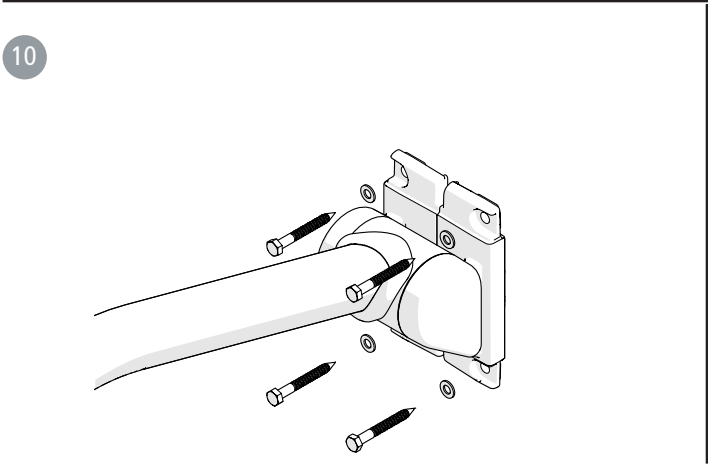11

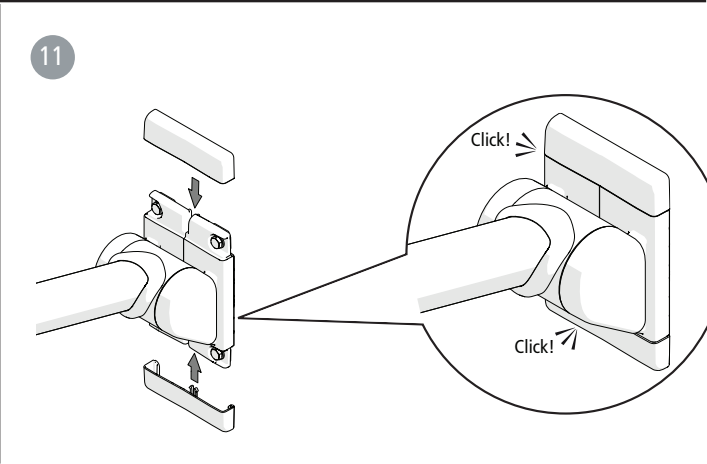12

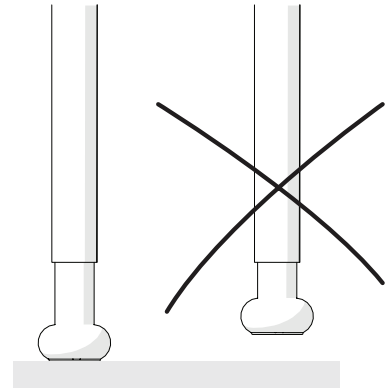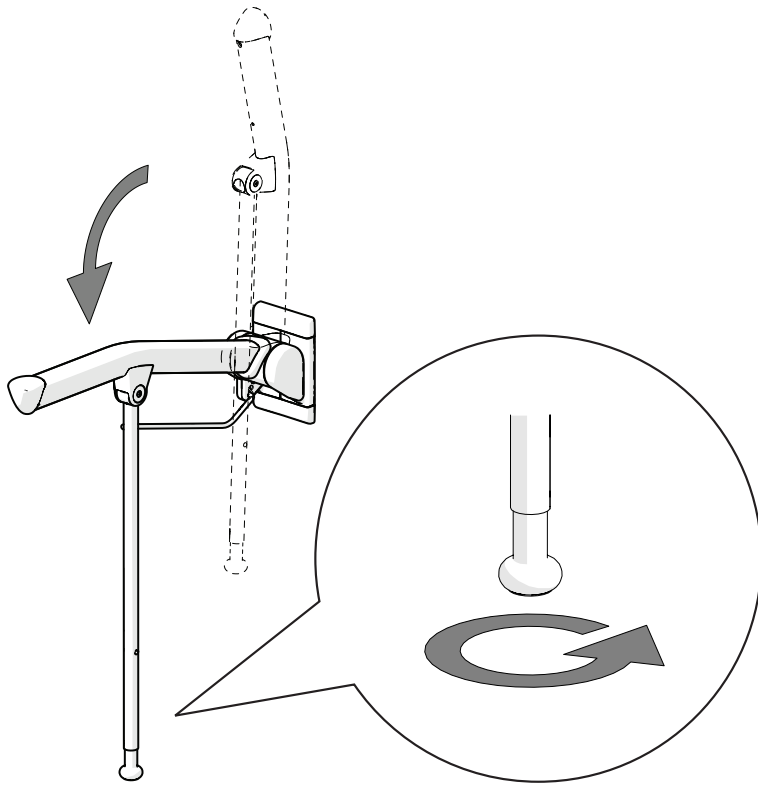C

1

2

3

D

1


2


3


4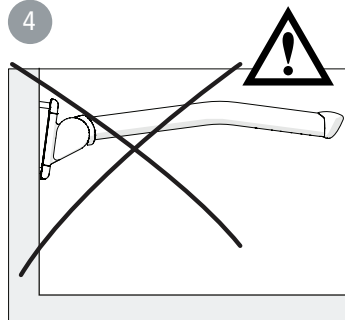


5

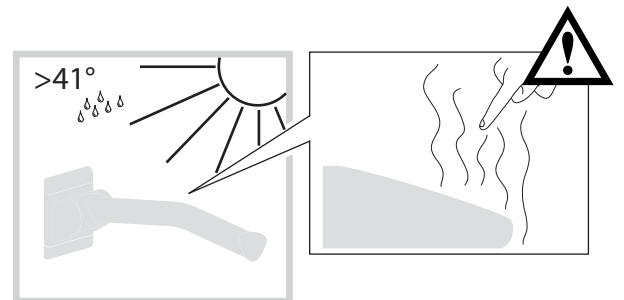E

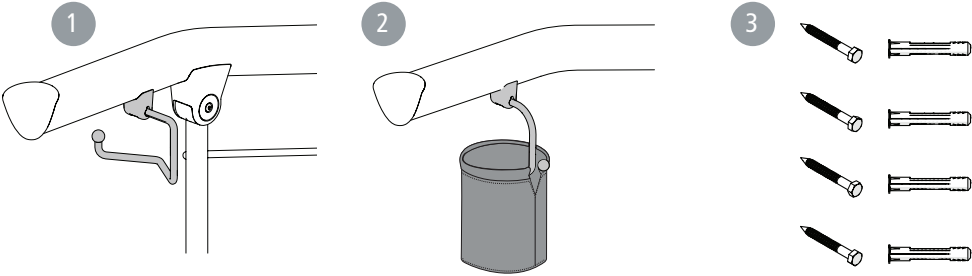F

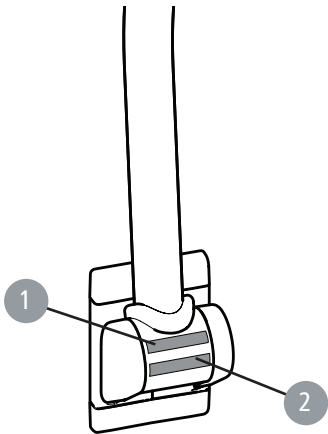G

600/700/850 mm (23½"/27½"/33½")	670/767/911 mm (26½"/30¼"/33½")	155 mm (6")	155 mm (6")	130 mm (5")	155x95 mm (6"x3¾")	193x130 mm (7½"x5")	800 +70 -30 mm (31½" +2¾" -1¼")	*135 kg/297 lbs	150 kg/330 lbs

*85 cm=100 kg/220 lbs

Etac Rex 60 cm	Etac Rex 70 cm	Etac Rex 85 cm	Etac Rex 70 cm	Etac Rex 85 cm		
2.7 kg 6 lbs	2.9 kg 6.4 lbs	3.3 kg 7.3 lbs	3,6 kg 7.9 lbs	4,0 kg 8.8 lbs	Alu/ABS/PUR/PP/SS	PA6GF/TPE/SS

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Tekniske data Figur G**English****General, intended use**

Thank you for choosing a product from Etac. To avoid injury during installation, handling and use, please ensure that you read this manual thoroughly and keep it. You can also find it at www.etac.com, where you will also find other documentation, including prescriber information.

The Etac Rex is a wall-mounted toilet arm support designed as a standing support for toilet use by persons with restricted mobility. The Etac Rex toilet arm support can be folded up when not in use. The Etac Rex toilet arm support with support legs is recommended where an arm support for user weights above 135 kg is required (max 150 kg). Etac Rex is not designed to be used in any other way than described above.

Etac Rex has been tested and complies with the requirements set out in EN ISO 10993-1, SS EN 12182.

Etac Rex is CE marked according to Directive 93/42/EEC on medical devices.

Delivery contents Figure A**Installation options**

May be mounted on walls with sufficient load-bearing capacity only. Please ensure that suitable fixtures are used. For mounting in concrete, plywood or wooden joists, we recommend the screw set accessory.

Mounting on walls with insufficient load-bearing capacity or with unsuitable fixtures can affect user safety. Mounting of the product and assessment of the wall's load-bearing capacity must be carried out by a qualified person.

Installation..... Figure B
Usage Figure C
Maintenance and safety Figure D

1. Clean/disinfect the product with normal non-abrasive cleaning agents with a pH value of between 5-9, or with 70% disinfectant solution. Rinse and dry.
The product is otherwise maintenance-free.
2. The product can withstand cleaning in a sterilizer at 85°C for 3 minutes. This applies for reconditioning.
3. Maximum user weight (see technical data).
4. Check that the bolts are securely tightened. Defective or incorrectly installed products must not be used. In case of damage, contact your dealer.
5. Heated product can cause burns.
The materials used are corrosion-resistant.
The materials used are resistant to all general disinfectants.

Guarantee, service life

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 10 years. For complete information regarding the service life of the product, see www.etac.com.

Accessories..... Figure E

1. Toilet paper holder 2. Storage basket 3. Screw set
Label placement Figure F

1. Product label (warnings, CE-marking)
2. Item no label (serial no, production date)

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R82 UK Limited.
Snug Seat, Inc.
Etac Supply Center AB